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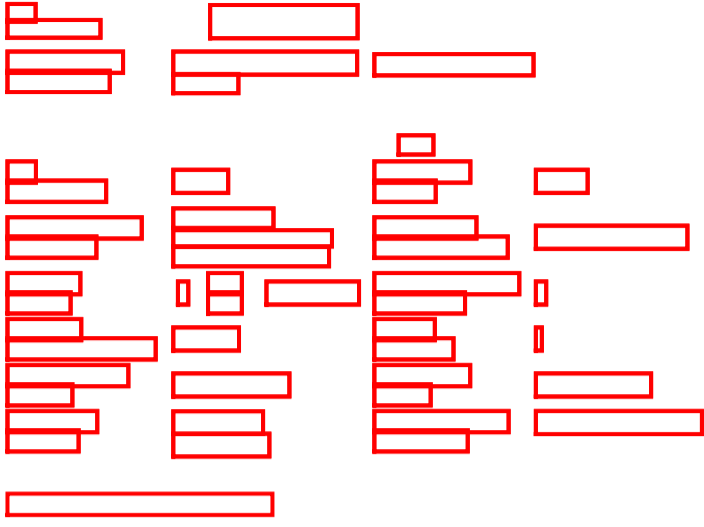
Please read these instructions for use carefully – they contain important information on how to use this product. If you have any questions, please ask your pharmacist, your doctor or contact us.

**BAUSCH + LOMB**

EN

## Artelac® Complete

**Composition:** 0.24% hyaluronic acid (as sodium hyaluronate), carbomer, glycerol, lipid component, sodium hydroxide and water for injection

**Intended use:** Eye lubricant

### What is Artelac Complete?

**Artelac Complete** is an opaque eye drop solution that contains 0.24% hyaluronic acid (as sodium hyaluronate), carbomer, glycerol and a lipid component.

The advanced formulation possesses a very high tolerability as it is free of preservatives and free of buffers.

### What is the application of Artelac Complete?

**Artelac Complete** is an eye lubricant that acts on all three layers of the tear film to improve the moistening of the ocular surface in the presence of dry eye sensation.

### When you should use Artelac Complete?

**Artelac Complete** is inspired by the design of a natural tear film. It contains the natural moisturizer hyaluronic acid (as sodium hyaluronate), which is found in the structures of a healthy human eye and the natural tear film and two additional moisturizers: Glycerol and Carbomer. Containing an additional lipid-component (medium chain triglycerides) **Artelac Complete** acts on all three layers of the tear film: lipid, aqueous and mucin phase.

**Artelac Complete** is designed to remain on the surface of the eye providing moisturization for immediate, long-lasting symptom relief. It is a suitable product for everybody suffering from a range of symptoms of dry eyes such as tired, stressed, gritty, burning and/or tearing eyes.

Dry eyes are caused by

- external factors, like extensive computer use, air conditioning, heating, cigarettes smoke, as well as windy outdoor environments as well as wearing contact lenses.
- Other factors, like certain drugs (such as the contraceptive pill, antihistamines, betablockers, antispasmodics and diuretics) or certain diseases that play a role in the development of ocular dryness.

### How often and how long should you use Artelac Complete?

Instill one drop of **Artelac Complete** into the eye as the need arises. You can use it every day and as often as you want. There is no limit of duration of use.

Discard the single dose unit after use.

### How to use Artelac Complete?

 <p><b>1</b> Separate one single-dose unit from the strip carefully.</p>	 <p><b>3</b> Lean your head back slightly and gently pull your lower eyelid down with a finger of your free hand. Hold the single-dose unit as vertically as possible above your eye with the nozzle pointing downwards. Make sure that the single-dose unit does not touch your eye to prevent any possible injury. Then gently squeeze the single-dose unit to apply one to two drops.</p>
 <p><b>2</b> Twist (don't pull) the top cap off the single-dose unit.</p>	

Close your eye afterwards and slowly roll it to optimally spread the drop on the surface of your eye.

### When you should not use Artelac Complete?

Do not use this product if you are hypersensitive to any of the ingredients.

### Important information about using Artelac Complete

- Local hypersensitivity reactions have been very rarely reported. Treatment should discontinue in such cases. If you experience persistent discomfort or irritation, stop using the product and consult your doctor.
- Do not touch the eye surface with the tip of the single dose to prevent any possible injury.
- Remove contact lenses before usage.
- Wait approximately 15 minutes before using another eye product or before replacing contact lenses.
- This product may cause temporary blurred vision after instillation which can have negative impact on driving vehicles or operating machines.
- Do not re-use.
- Reuse of the product bears a potential risk of infection to the user
- Do not use after the use-by date.
- Do not use if the package or single-dose unit is damaged.
- **Artelac Complete** is filled in small plastic units that could be swallowed by small children. Danger of suffocation. Therefore, **Artelac Complete** should be kept out of reach of children.
- This device can be used by lay persons not specifically trained.
- Based on current knowledge, the product can be used for pregnant, breastfeeding women and children.
- If your child is experiencing dry eyes, consult a doctor before using an eye lubricant.

### How to store Artelac Complete?

Store at a temperature of +1 to +25°C.  
Keep away from sunlight.

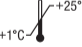
**Disposal:** The product must be disposed of in accordance with local regulations.

Every serious incident that has occurred in relation to the product should be reported to the manufacturer and the competent authority.

**Presentation:** **Artelac Complete** is available in packs containing 30 single dose units with 0.5 ml eye drops.

This product is also available in preservative-free multi-dose bottles.

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**Artelac Complete, I211EU-05; national revision (2023/05)**

<sup>®/™</sup> are trademarks of Bausch & Lomb Incorporated or its affiliates

© 2023 Bausch & Lomb Incorporated or its affiliates

P1GR05

42570PB211/S-GR