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Please read these instructions for use carefully – they contain important information on how to use this product. If you have any questions, please ask your pharmacist, your doctor, eye care professional or contact us.

BAUSCH + LOMB

Recugel®

en

Composition: 1 g of eye gel contains 50 mg of dexpanthenol (medicinal substance) as well as carbomer, disodium edetate, sodium hydroxide, water for injection and cetrimide as preservative agent.

Intended use: Viscous solution with Dexpanthenol to improve the lubrication of the ocular surface and to protect the healthy eye.

Application of RECUGEL


RECUGEL has a refreshing and soothing effect on the eyes. It improves the moistening of the ocular surface in the presence of dry eye sensation, burning or tiredness of the eyes, caused, for instance, by watching television for a long time or extensive VDU work, air from heating systems or air-conditioning.

RECUGEL contains the substance dexpanthenol for protection of healthy eyes. Dexpanthenol is the provitamin of Vitamin B5.

How often and how long should you use RECUGEL?

Instill 1-2 gel drops of RECUGEL into the eye. It can be used up to 4 times daily.

Safe and correct use of RECUGEL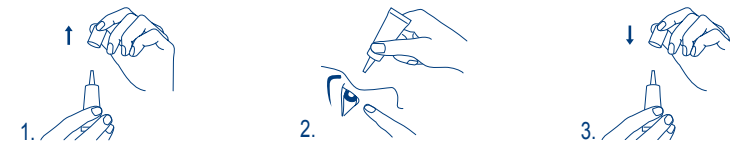

1. Remove the protective cap from the tip of the tube before each use.
2. Lean your head back slightly and pull your lower eyelid down slightly with a finger of your free hand and hold the tube vertically above your eye with the nozzle pointing downwards. Then gently squeeze the tube to apply one gel drop.
3. Replace the cap on the tube after each use.

Close your eye and move your eyeball slowly from one side to the other to optimally spread the gel drop on the surface of your eye.

When you should not use RECUGEL?

Do not use this product if you are hypersensitive to any of the ingredients. Do not use with concurrent therapy of neurotrophic keratitis with Oxervate.

Important information about using RECUGEL

- Local hypersensitivity reactions have been very rarely reported. Treatment should be discontinued in such cases. If you experience persistent discomfort or irritation, stop using the product and consult your doctor.
- This product may cause temporary blurred vision after instillation which can have negative impact on driving vehicles or operating machines.
- Do not touch the eye surface with the tip of the tube to prevent any possible injury of the eye surface.
- Avoid contact of the tip of the tube with your eye or your fingers to prevent any possible contamination of the gel.
- Once opened, do not share the tube with anyone else.
- Remove contact lenses before usage.
- Wait approximately 15 minutes before using another eye product or before replacing contact lenses.
- Do not use after the use-by date.
- Do not use if the tube is damaged.
- RECUGEL contains small plastic parts that could be swallowed by small children. Danger of suffocation. Therefore, RECUGEL should be kept out of reach of children.
- This device can be used by lay persons not specifically trained.
- Prior to use in children, during pregnancy and while nursing, consult a doctor before use.

How to store RECUGEL

Store the medical device in a temperature range of +1 °C to +25 °C. Discard the medical device after 6 weeks once opened.

Disposal:

The product must be disposed of in accordance with local regulations. Every serious incident that has occurred in relation to the product should be reported to the manufacturer and the competent authority.

The summary report of safety and clinical performance is available in the European database on medical devices (EUDAMED): <https://ec.europa.eu/tools/eudamed>

Basic UDI: 4030571B030074


Presentation:

RECUGEL is available in packs of 10 g tubes.

Explanation of Symbols

Sterilized using aseptic processing techniques and packaged in a single sterile barrier system

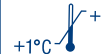Temperature limit

Batch code

Discard the medical device 6 weeks after first opening

Contains a medicinal substance (contains dexpanthenol)

Manufacturer
Dr. Gerhard Mann chem.-pharm.
Fabrik GmbH
Brunsbütteler Damm 165/173
13581 Berlin, Germany

European Medical Device Conformity Assessment
CE mark with notified body number

Do not use if package or tube is damaged

Caution (consult instructions for use for important cautionary information)

Use-by date

Medical Device

Consult instructions for use

Date of manufacture

Unique Device Identification
GTIN (01) device identification
EXP (17) use-by-date (format: yymmdd)
LOT (10) lot number

Date of issue: RECUGEL, "I300EU-04"; national revision date (2023-04)

®/™ are trademarks of Bausch & Lomb Incorporated or its affiliates

© 2023 Bausch & Lomb Incorporated or its affiliates